User's Guide

for the electronic magnifier *eMag* 43 HD

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Congratulations on purchasing your eMag 43 HD. This electronic handheld magnifier allows you to magnify texts and objects from 1.6 to 14 times original size. The eMag 43 HD has a LCD screen (16:9 aspect ratio) measuring 10.9 cm (4.3") diagonally. Enjoy this high-quality device.

The information and data contained in this document is subject to change without notice. Without the express prior written consent of A. Schweizer GmbH, no part of this document may be reproduced or transmitted for any purpose, in any form or by any means, electronic or mechanical.

Copyright © Januar 2014 A. SCHWEIZER GmbH. All rights reserved.

Contents

| | | | |
|---|-----------|---------------------------------------|-----------|
| Please read these instructions first | 6 | Using the Line Marker or Reading Mask | 22 |
| Overview | 7 | Displaying the Line Marker | 22 |
| Your eMag 43 HD | 8 | Displaying the Reading Mask | 23 |
| Quick Start | 9 | Storage Mode | 24 |
| LED-Indicator "Battery Status" | 11 | Icons used in Storage Mode | 24 |
| Inserting Batteries/Rechargeable Batteries | 12 | Saving and Deleting an Image | 25 |
| Charging Rechargeable Batteries | 13 | Viewing Saved Images | 25 |
| Description of Control Buttons | 14 | Saving and Deleting an Image when | |
| Turning on the eMag 43 HD | 15 | Memory is full | 26 |
| Functions | 16 | Transferring Images to a Computer | |
| Magnifying Live Images | 16 | via USB Cable | 27 |
| Magnifying Frozen and Saved Images | 16 | Transferring Images to the eMag 43 HD | |
| Changing Colour Mode | 17 | via USB Cable | 28 |
| Switching from Last Used Colour Mode | | Displaying a Slide Show | 29 |
| to Full Colour | 17 | Troubleshooting | 30 |
| Displaying more Colour Modes | 18 | Specifications | 33 |
| Resetting Standard Colour Modes | 19 | Technical Standards | 34 |
| Colour Modes | 19 | Safety and Maintenance | 34 |
| Standard Colour Modes | 19 | Cleaning the Casing of the eMag 43 HD | 35 |
| Optional Colour Modes | 20 | Cleaning the LCD Screen | |
| Freezing an Image | 21 | of the eMag 43 HD | 35 |
| Turning LED Lights off | 21 | | |

Please read these instructions first

The four rechargeable AAA batteries (NiMH rechargeable batteries, 1000 mAh) must be charged prior to using the eMag 43 HD for the first time. These rechargeable batteries are included. They will be fully charged after approx. 2 hours.

You can also use 4 AAA disposable alkaline batteries to power the unit.

Caution: Do not use rechargeable batteries (NiMH) together with non-rechargeable alkaline batteries. Always replace all batteries with new ones of the same chemistry. Do not mix new and old batteries. Replace rechargeable batteries (NiMH) with 1000 mAh NiMH rechargeable batteries. Failure to follow this precaution can result in battery failure or leakage of battery fluid. Replacing batteries with the wrong type may cause bursting. Do not use batteries that specify a maximum charging current below 250 mA.

Refer to **Charging Rechargeable Batteries** → on page 13 for instructions on battery charging.

Refer to **LED Indicator “Battery Status”** → on page 11 for descriptions of the battery indicator.

For the basics you will need for getting started with the eMag 43 HD, refer to **Quick Start** → on page 9.

For a general **overview** of the eMag 43 HD you should read this guide first.

Overview

Thank you very much for choosing the eMag 43 HD!




The eMag is a portable, electronic handheld magnifier that can **magnify objects from 1.6 to 14 times original size**. It integrates a HD-camera providing sharp **HD quality** images that can be easily viewed on its full colour, **4.3 inch LCD screen**. The eMag 43 HD supports 5 Standard Colour Modes. 15 additional high **contrast colour viewing modes** can be activated easily.

The eMag 43 HD has a **fold-out handle** that allows you to hold the magnifier close to an item. Press the magnification button for continuous zoom (1.6x to 14x with the handle extended or 4x to 11x with the handle folded in), and then freeze the image on the screen. If you wish, you can then adjust the **magnification**. This is ideal for viewing clothing labels, recipes, price tags, receipts, business cards etc..

The eMag 43 HD can **save** and store up to **80 images** that can easily be transferred to a computer via a USB connection.

The eMag 43 HD also has a **line marker and reading mask** function you can activate as required and that helps you to quickly find and keep your place while viewing an image or reading text on the LCD screen.

Your eMag 43 HD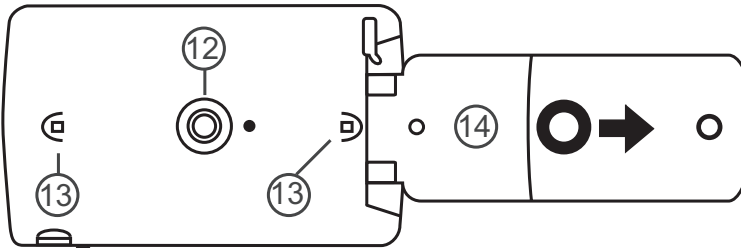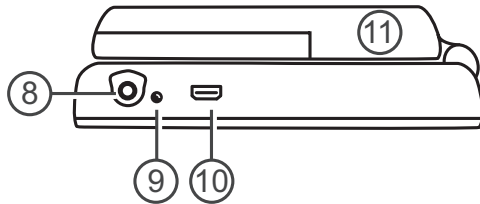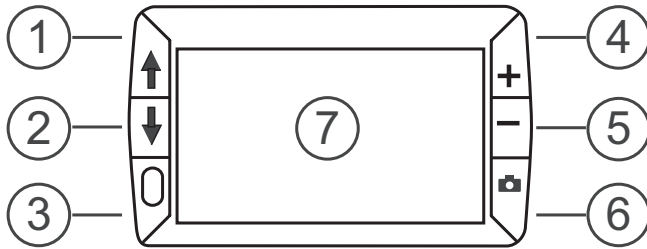


1. Colour Select Scroll Up (blue)
2. Colour Select Scroll Down (blue)
3. Power Button (green / yellow)
4. Magnification Increase (yellow)
5. Magnification Decrease (yellow)
6. Freeze Frame (red)
7. LCD Screen
8. Power Connector
9. LED Display "Battery Status"
10. USB Port (micro USB)
11. Handle and Battery Compartment in closed position
12. HD Camera
13. LED lights
14. Handle and Battery Compartment in open position

Quick Start

The following steps will show you how to quickly set up and use the eMag 43 HD for the first time.

1. Make sure the **unit is charged**. -> Refer to page 13.
2. **Turn on the eMag 43 HD** by pressing the green-yellow button  -> Refer to page 15.
3. Proceed with one of the following options:
 - **Unfold the handle** in order to hold and use the eMag 43 HD like a magnifying glass for enlarging and reading labels, or small print on cans, packages, medicine labels etc.
 - Lay the eMag 43 HD **with the handle folded** in on a flat document, such as a newspaper, magazine, bill, letter etc.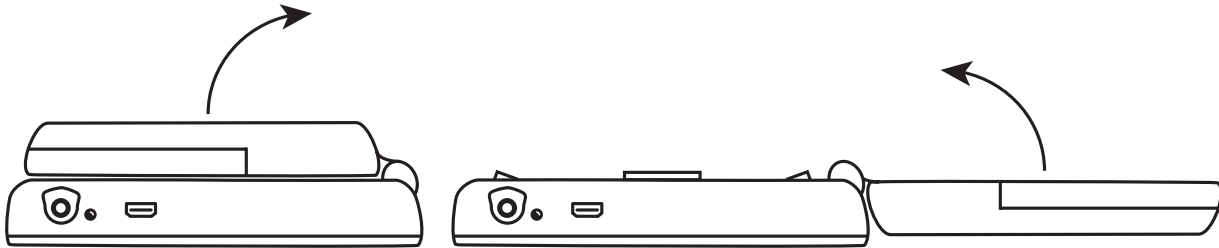

4. Adjust the **magnification** using the yellow buttons (**+** or **-**). -> Refer to page 16.
5. Select a **colour mode** (**↑** or **↓**) for contrasting colours to suit your most comfortable reading.
-> Refer to pages 17-20.
6. **Freeze the image** on the LCD screen , so you can take your time to view the details in the image.
-> Refer to page 21.
7. **Save the image or return to live view** . -> Refer to page 25.

When you are familiar with the basics of the eMag 43 HD, you can try the following **advanced functions**:


- Turn the **LED reading lights on and off**, to avoid glare and reflections when viewing photographs or glossy magazines. → Refer to page 21.
- Use the **line marker or the reading mask** features for keeping your place while reading text.
→ Refer to pages 22/23.
- You can **save, view and delete images** stored on the eMag 43 HD.
→ Refer to pages 25/26.
- For additional storage **transfer the images from the eMag 43 HD to your computer**.
→ Refer to page 27

LED Indicator “Battery Status”

- Green – Solid:** Battery charged – power adapter connected.
- Green – Flashing:** Battery charging – power adapter connected.
- Red – Flashing:** Low battery status.
- Red – Solid:** Error; Unplug the power adapter, wait a few seconds and plug it in again. If the indicator stays on, please replace the batteries.
- No LED:** Battery charged, the power adapter is not plugged in or the eMag 43 HD is off.


Inserting batteries / rechargeable batteries

1. Unfold the handle completely and remove the battery cover by sliding the cover in the direction of the arrow.
2. Insert the batteries / rechargeable batteries, close the battery cover and fold the handle under the unit. Insert the batteries / rechargeable batteries observing the correct polarity –as shown in the illustration below: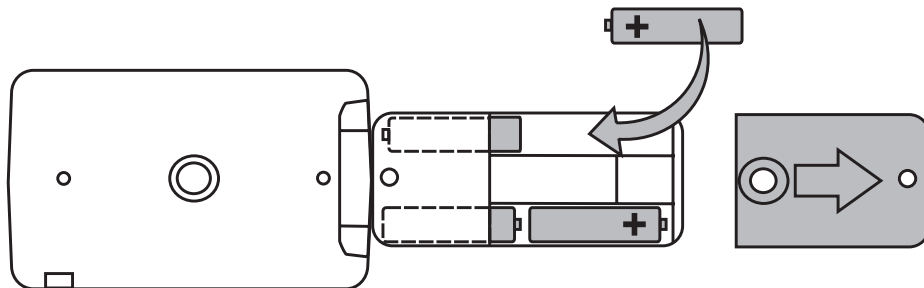

Charging rechargeable batteries

1. Plug one end of the power adapter into the eMag 43 HD and the other end into a standard AC outlet (1).
2. The LED indicator (2) blinks green to show that the rechargeable batteries (NiMH) are charging.
For more information on **LED Indicator Battery Status** -> Refer to page 11.

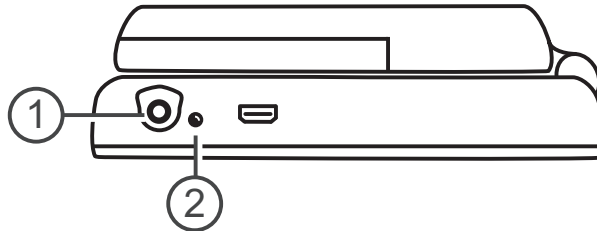3. When the battery is charged, the LED indicator stops blinking and glows a steady green. Unplug the power adapter from the eMag 43 HD. The LED Indicator “Battery Status” turns off.
4. You can now use the eMag 43 HD continuously for approx. 2 hours. When the battery/rechargeable battery is almost drained, the LED indicator “Battery Status” flashes red. Connect the power adapter and plug it in to recharge the rechargeable battery (or insert new batteries).

Note: If you wish to replace the supplied rechargeable batteries, replace all four at the same time with **1000 mAh NiMH rechargeable batteries** with a MAXIMUM charging current above 250 mA.

Description of Control Buttons

Power (Green/Yellow):  Press to turn power on, press again to turn power off.

Magnification (Yellow):  Press and HOLD for continuous zoom magnification with live view. Press and release to increase or decrease the magnification of frozen images in increments.

Colour Select (Blue):  Press to cycle through standard colour settings and 15 additional colour modes. For a complete overview -> Refer to pages 19/20.

Freeze Frame (Red):  Press to freeze the current image. Press again to turn off Freeze Frame Mode and to return to live view.

Turning On/Off the eMag 43 HD

1. Press the green-yellow button  to turn on the eMag 43 HD. After a few seconds the icon for the battery status is displayed on the screen.
2. Press the button again to turn off the eMag 43 HD.

Note: If the eMag 43 HD is on, but no buttons are pressed, it will turn off automatically after 30 minutes – this feature extends battery life.

Functions

Magnifying Live Images

When viewing live images you can use the unit with the handle extended for continuous zoom magnification from 1.6 to 14 times or with the handle folded in for magnification from 4 to 11 times.

- Press and HOLD the yellow PLUS button **+** to increase magnification.
- Press and HOLD the yellow MINUS button **-** to decrease magnification.

Magnifying frozen and saved images

You can freeze and save an image displayed on the LCD screen. When viewing a frozen or a saved image you can increase or decrease the magnification in increments by pressing the yellow buttons (**+** or **-**).

For instructions on freezing an image refer to **Freezing an Image** -> on page 21.

To access stored images, refer to **Saving and Deleting an Image** -> on page 25.

Changing Colour Modes

A colour mode is a combination of pre-set colours (such as yellow text on a blue background) you can select yourself when using the eMag 43 HD.

The eMag 43 HD has 5 standard colour settings and 15 additional colour modes. For a complete list, refer to **Colour Modes** → on pages 19/20.

Press the blue arrow buttons (↑ or ↓) to change the current colour combination on the LCD screen. The blue UP arrow button ↑ cycles in one direction through the list of colour combinations while the blue DOWN arrow button ↓ cycles through the list in the other direction.

Switching from last used Colour Mode to Full Colour

The eMag 43 HD has a convenient feature that allows you to quickly switch from the last used colour mode to full colour – or the other way around. This is a useful shortcut if you are viewing a live image (for instance while reading and holding a document) and have selected a colour mode offering the best possible contrast for you. If you have to move the eMag 43 HD to another part of the document, you might find it easier to switch to full colour, move to the new position and then to switch back to the colour mode you selected beforehand and continue reading. To do this, follow these steps:

- Press and HOLD the green-yellow button  and press the blue DOWN arrow button ↓. The screen switches from one mode to the other.
- To switch back, repeat the same button sequence.

Displaying more Colour Modes

The eMag 43 HD has 5 Standard Colour Modes. You can also choose and activate one of the 15 other color modes. As soon as a colour mode is available, you can switch to it easily using the blue arrow buttons (↑ or ↓).

To add or remove further Colour Modes:

1. Press and HOLD the green-yellow button  and then press the blue UP arrow button ↑ to enter Colour Selection Mode.
The enable checkbox is checked to indicate that the current colour mode is active.
2. To disable the current colour mode, press the blue DOWN arrow ↓ while STILL holding DOWN the green-yellow button . This button sequence clears the checkbox .
3. To select the next colour mode, press the blue UP arrow ↑ while STILL holding DOWN the green-yellow button .
 - If the enable checkbox is checked , the colour mode is active.
 - If the enable checkbox is cleared , the colour mode is disabled.
4. To enable or disable a colour mode, press the blue DOWN arrow ↓ while still holding down the green-yellow button  to select or clear the checkbox .
5. Repeat steps 3 and 4 to add or remove other colour modes.
6. When you are finished with selecting colour modes, release all buttons. Any colour mode alteration will be saved, and the unit will return to Standard Operating Mode.

Resetting Standard Colour Modes

If you have added or removed Optional Colour Modes, you can always reset the eMag 43 HD to the 5 Standard Colour Modes.

Simultaneously press and HOLD DOWN the green-yellow button  and the blue UP arrow  for 15 seconds.

Colour Modes

There are 5 Standard Colour Modes and 15 Optional Colour Modes you can activate yourself.

Standard Colour Modes

1. True colour (always available, cannot be disabled)
2. White on Black
3. Black on White
4. Yellow on Blue
5. Yellow on Black

Optional Colour Modes

- Black on Yellow
- Green on Black
- Black on Green
- White on Blue
- Black on Blue
- Blue on Black
- Blue on Yellow
- Red on Black
- Black on Red
- Magenta on Black
- Black on Magenta
- Purple on Black
- Black on Purple
- Red on White
- Black on Light Black

Freezing an Image

Freezing an image on the LCD screen can be useful for reading labels on objects or supermarket shelves. You can hold the eMag 43 HD close to an object, freeze the image and then hold the eMag 43 HD close for comfortable reading.

1. Press the red button  to freeze an image.
If you wish, press the yellow PLUS button  to increase magnification or the yellow MINUS button  to decrease magnification.
2. Press the red button  once more to exit the Freeze Frame Mode and to return to live view.

Note: To save a frozen image, press and HOLD the red button  for 3 seconds.
Refer to **Saving and Deleting an Image** -> on page 25 for further information.

Turning off the LED lights

When you turn on the eMag 43 HD, the LED lights are turned on as well. To avoid glare when viewing reflective surfaces, you can turn off the LED lights:


1. Press and HOLD the green-yellow button  and then press the red button . The LED lights turn off.
2. To turn the LED lights back on, press and HOLD the green-yellow button  and then press the red button .

Using the line marker or reading mask

The line marker is a horizontal line that appears on the screen. The reading mask covers certain parts of the screen in a way that only a horizontal window is displayed on the screen. Use these features for better orientation and focusing when viewing an image. If you use the button sequence as described below, the eMag 43 HD will first display the line marker, then the reading mask and after that neither of the two.

Displaying the line marker


1. Press and HOLD the green-yellow button  (1) and then press both yellow buttons (+ and - (2 and 3)). The line marker appears on the screen as shown in the following illustration:

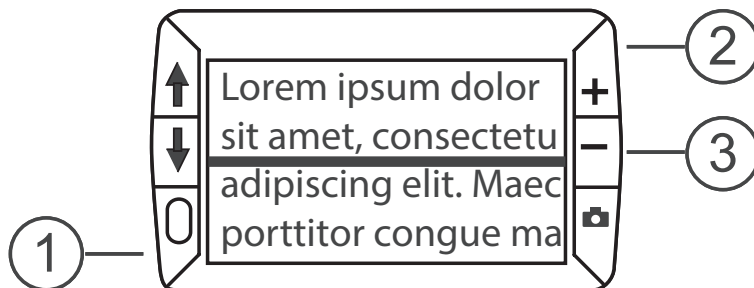2. Press and HOLD the green-yellow button  (1) and then press the yellow PLUS button + (2) or the yellow MINUS button - (3) to shift the line marker up or down.

Displaying the Reading Mask

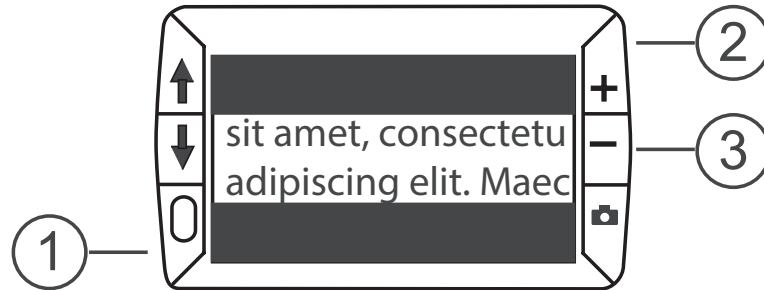1. Press and HOLD the green-yellow button  (1) and then press both yellow buttons (**+** and **-** (2 and 3)) to remove the line marker and to display the reading mask on the screen as shown in the following illustration:

2. Press and HOLD the green-yellow button  (1) and then press the yellow PLUS button **+** (2) or the yellow MINUS button **-** (3), to increase or decrease the distance between the masks.
3. To remove the reading mask from the screen, press and HOLD the green-yellow button  (1) and then press and HOLD both yellow buttons (**+** and **-** (2 and 3)).

Storage Mode

Storage Mode allows you to save up to 80 images. When first saving an image, it is saved with the currently displayed magnification and colour settings. When **viewing a stored image** you can **alter the magnification level**. -> Refer to page 16.

GB

Storage Mode Icons

- Save:** Simultaneously press the 2 yellow buttons (**+** and **-**) to activate the checkbox. The image will be saved when you exit Storage Mode.
- Delete:** Simultaneously press the 2 yellow buttons (**+** and **-**) to clear the checkbox . The image will be deleted when you exit Storage Mode.
- Memory Full:**  Indicates that the maximum number of images has been saved on the eMag 43 HD. Before you can save the current image you will have to mark one or more images for deletion.

Saving and Deleting an Image

After freezing an image (→ page 21) you can **save it** by following these steps:

1. Press and HOLD the red button  for 6 seconds to enter Storage Mode.
2. The marked checkbox "save" is automatically displayed on the screen. It indicates that the image will be saved after exiting Storage Mode and returning to Standard Mode.

To delete an image in Storage Mode:

1. Press and HOLD both yellow buttons (**+** and **-**) to clear the checkbox . When you exit Storage Mode, the image is deleted.
2. Press and HOLD the red button  for 3 seconds to exit Storage Mode and to return to live view.

Viewing saved images

The eMag 43 HD can store up to 80 images. You can browse through them easily using the blue arrow buttons (**↑** and **↓**).

1. Press and HOLD the red button  for 6 seconds to enter Overview Mode.
 - To browse forward through the saved images, press the blue UP arrow **↑**.
 - To browse backward through the saved images, press the blue DOWN arrow **↓**.

Note: To quickly browse through the images, press and HOLD one of the blue arrow buttons (**↑** or **↓**). Release the button to stop browsing.

2. Press and HOLD the red button  for approx. 3 seconds to exit this mode and to return to live view.

Saving and Deleting an Image when Memory is full

The eMag 43 HD can store up to 80 images. If you try to add another image, the “Memory Full” icon  appears on the screen. To add another image, you first have to delete one or more saved images from the memory.


1. Press the blue UP arrow  or the blue DOWN arrow  to navigate to the image you wish to delete.
2. Press both yellow buttons ( and ) to clear the checkbox .
3. Press and HOLD the red button  for 3 seconds to exit Storage Mode. The new image is saved and all unchecked images are deleted.

Note: To create memory space on your eMag 43 HD you can copy images to your computer.
Also refer to **Transferring Images to a Computer via USB Cable** -> on page 27.

Transferring Images to a Computer via USB Cable

Images stored on the eMag 43 HD can be transferred to a computer via the supplied USB cable with the following steps:

1. Connect the supplied USB cable to the eMag 43 HD (1) and the computer.

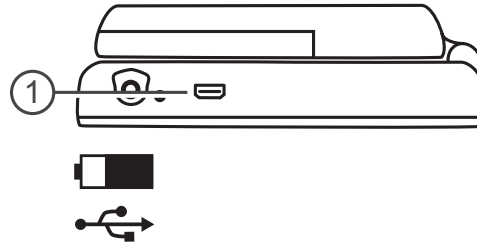2. When connected,
 - The USB and battery icons appear briefly on the LCD screen.
 - The battery icon indicates the current battery status. The eMag 43 HD does not charge and no images are displayed on the screen while connected to the computer via USB.
 - The eMag 43 HD shows up as an external device in Explorer (Windows) or Finder (Mac).
3. Open the “eMag 43 HD DCIM” directory and use the “drag and drop” method to move or copy one or more images to a folder or your computer desktop. The images are named and numbered consecutively, e.g. IMG_0001.bmp, IMG_0002.bmp etc. in the saved file.
4. When you have transferred your files, make sure to remove the eMag 43 HD safely from your computer.

Note: To disconnect the eMag 43 HD safely and to protect your data, follow your operating system’s instructions on safe removal of external USB devices.

Transferring Images to the eMag 43 HD via USB cable

Every image, that has been transferred to your computer from the eMag 43 HD, can be easily transferred back to your eMag 43 HD.

Note: Only images that were saved on the eMag 43 HD can be copied back to the eMag 43 HD.

1. Make sure that the eMag 43 HD is connected to your computer with the supplied USB cable.
2. Find the images on your computer using Explorer (Windows) or Finder (Mac).
3. Select the image and drag it to the “eMag 43 HD DCIM” directory.
4. When you have transferred your files, make sure that you remove the eMag 43 HD safely from your computer.

Note: To disconnect the eMag 43 HD safely and to protect your data, follow your operating system’s instructions on safe removal of external USB devices.

Displaying a Slide Show

You can create a slide show that presents the images saved on the eMag 43 HD in an infinite loop. Each image is displayed for approx. 5 seconds. The images are shown in an order determined according to their file names, such as IMG_0001.bmp, IMG_0002.bmp etc.

To change the order in which the images appear, you can transfer them to your computer, rename the images according to the desired order and then transfer them back to your eMag 43 HD. (The file names should not be longer than 8 characters plus the file extension.)

For detailed instructions on transferring files from the eMag 43 HD to the computer, refer to **Transferring Images** -> pages 27/28.

Note: In Slide Show Mode, the eMag 43 HD does NOT turn off automatically after 30 minutes.

Follow these steps to **start the slide show**:

1. Press and HOLD the red button  for approx. 6 seconds to enter Review Mode.
2. Press and HOLD the green-yellow button  and then press the blue UP arrow .
The first (depending on its file name) saved image is displayed on the LCD screen for approx. 5 seconds, then the second image, then the third one etc.
3. Press and HOLD the red button  for approx. 3 seconds to exit Slide Show Mode and to return to live view.

Troubleshooting

| Problem | Solution |
|--|--|
| The unit will not turn on. | <ul style="list-style-type: none">• Press the green-yellow power button. If the eMag 43 HD does not turn on, charge the NiMH batteries or replace the batteries. Note: The battery does not charge when the eMag 43 HD is connected to the USB port.• Make sure the batteries are inserted correctly. → page 12. |
| The screen is blank. | <ul style="list-style-type: none">• If the eMag 43 HD is connected to the computer via USB, the screen is blank.• Turn on the LED lights.• Turn off the eMag 43 HD and then turn it on again.• Make sure the item to be magnified is positioned correctly under the camera window.• Press the yellow magnification button to step through the magnification levels.• Press the blue Colour Select button to view the various colour combinations. |
| The LED indicator “Battery Status” is flashing red. | <ul style="list-style-type: none">• Charge the rechargeable NiMH batteries or replace the AAA disposable alkaline batteries. (Do not mix battery chemistries.)• When replacing rechargeable batteries, use only 1000 mAh NiMH rechargeable batteries. |
| The LED indicator “Battery Status” is solid red although the power adapter is connected to the eMag 43 HD. | <ul style="list-style-type: none">• A battery error has occurred. Unplug the power adapter from the eMag 43 HD and plug it in again. (continued on next page) |

| Problem | Solution |
|--|--|
| <p>The LED indicator “Battery Status” is solid red although the power adapter is connected to the eMag 43 HD.</p> | <ul style="list-style-type: none"> • Charge the NiMH batteries for at least 2 hours or replace them with AAA disposable alkaline batteries. Do not charge the disposable alkaline batteries. • If the problem persists, disconnect the power adapter and call your eMag dealer. |
| <p>The LED indicator “Battery Status” does not flash green, although the power adapter is connected and plugged into a power outlet.</p> | <ul style="list-style-type: none"> • Make sure the small end of the power adapter is fully inserted into the power connector of the unit. • Ensure that the power outlet is supplying power and functions properly. If the outlet is connected to a switch, make sure the switch is on. |
| <p>The NiMH batteries are not charging.</p> | <ul style="list-style-type: none"> • Make sure the NiMH batteries are inserted correctly. → page 12. (Do not replace batteries/NiMH batteries while the power adapter is connected. • Use the supplied power adapter to charge the eMag 43 HD. Please note that a USB port does not supply enough power to charge batteries. • Make sure the small end of the power adapter is fully inserted into the power connector of the unit. • Ensure that the power outlet is supplying power and functions properly. If the outlet is connected to a switch, make sure the switch is on. • Some rechargeable batteries have low charge rates and specify a “MAX Charging Current”. Do NOT use rechargeable batteries that specify a MAX Charging Current below 250 mA. |

Continued on next page

| Problem | Solution |
|---|---|
| The screen is stained or dirty. | Clean the LCD Screen as specified in "Cleaning the LCD Screen of the eMag 43 HD" -> page 35. |
| The text is too small or too large to read. | Press the yellow magnification button to increase or decrease magnification. For live images press and HOLD the magnification button for continuous zoom magnification. For frozen or saved images press the magnification button to increase or decrease the magnification in increments |
| LED lights cause glare on a reflective surface. | Press and HOLD the green-yellow power button and then press the red button (Freeze Frame) to turn off the LED lights. |
| After 30 minutes the eMag 43 HD turns off. | This is an intentional feature: the eMag 43 HD turns off automatically if no button has been pressed for 30 minutes. This feature extends battery life. |

Specifications

- **LCD Screen:** 109 mm diagonal (4.3")
- **Dimensions:** 126 mm (high) x 75 mm (wide) x 37 mm (deep)
- **Weight:** 228 g
- **Colour Modes:** 5 Standard Colour Modes (true colour, white on black, black on white, yellow on blue, yellow on black) as well as 15 additional colour modes
- **Magnification Range:** 1.6 x to 14 x, continuous zoom
- **Maximum Number of Saved Images:** up to 80 bitmap (BMP) images
- **Camera:** HD camera module
- **Batteries:** 4 rechargeable AAA NiMH batteries (1000 mAh), approx. 2 hours continuous use; Charging time approx. 2 hours
or 4 AAA disposable alkaline batteries

Note: Use only the supplied power adapter to charge the batteries of the eMag 43 HD.
Do NOT use batteries that specify a MAX charging current below 250 mA.

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GB

Safety and Maintenance

- Do not try to repair the eMag 43 HD yourself or to open its casing. This product does not contain any user serviceable parts. Any unauthorized attempt to service or replace internal components will void this product warranty..
- Keep the eMag 43 HD away from wet areas, rain and other sources of liquids.
Do not submerge the eMag 43 HD.
- Use the eMag 43 HD only in an area where the ambient temperature is between 10° and 35°.
- Do not store the eMag 43 HD in a location where extreme temperatures may occur, e. g. in a parked car.
- Do not attempt to force any other plug into the connector of an eMag 43 HD.
The power plug slides easily into the connector.

Cleaning the Casing of the eMag 43 HD

Ensure the eMag 43 HD is unplugged. Use a damp, soft, clean cloth with water or a mild cleaner. Do not use alcohol, solvents, abrasives or sprays.

Cleaning the LCD Screen of the eMag 43 HD

- Before cleaning the screen, turn off the eMag 43 HD and unplug the power connector.
- Always use a clean microfibre cloth.
- Gently wipe across the screen using the cloth. Do not press hard on the cloth or the LCD screen.
- When using a liquid cleaner, make sure it is suitable for a LCD screen. You can also use distilled water, isopropyl alcohol or vinegar mixed with water at a concentration of no more than 10%. Note that tap or mineral water may leave stains on the screen. Do not leave any liquid on the screen. Repeat this process for stubborn stains.
- DO NOT spray any cleaners directly on the LCD screen.
- DO NOT use cleaners containing acetone, ethyl alcohol, ethyl acetate, ammonia or methyl chloride.
- DO NOT use paper towels for cleaning the LCD screen.

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Copyright © Janvier 2014 A. SCHWEIZER GmbH. Tous droits réservés.
Copyright © Januari 2014 A. SCHWEIZER GmbH. Alle rechten voorbehouden.
Freedom Scientific GmbH